# evacare


## INSTALLATION GUIDE GRAB RAIL


#### **IMPORTANT**


- · For cavity walls, the grab rail must be fixed to the internal wall stud or a solid framing structure. This may require the grab rail to be installed on an angle to match the gap between the studs.
- The included fixings are for use with solid walls only. It is the installer's responsibility to ensure that the correct fixings are used to suit the wall type.


#### INSTALLATION


- 1. Slide the flange covers off to expose the bracket holes.
- 2. Place the grab rail in the desired position and mark the holes for drilling.
- 3. Using an appropriate drill bit (i.e. masonry bit for tiles), drill holes to suit the wall plugs.
- 4. Insert wall plugs.
- 5. Line up the grab rail with the drilled holes and install the screws.
- 6. Slide the flange covers over the brackets.

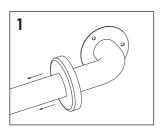


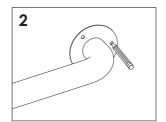

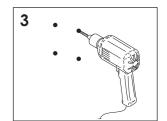

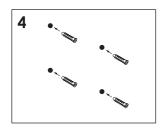

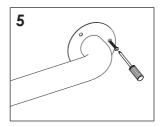

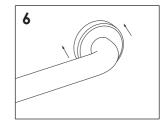

### CARE AND MAINTENANCE

- · Clean with a soft cloth and warm soapy water.
- · Do not use harsh detergents, corrosive chemicals, or abrasive cleaners.

#### **NOTES**

This product is manufactured to comply with international standards: ISO 17966:2016. This product also complies with the following national standards: AS 1428.1-2009, NZS 4121:2001.